

HOW TO ATTACH DRICUFF STRAP TO OVERCAST SHOWER COVER

Position the cover so that it faces away from you (the top is closest to you.) Place the square sticky Velcro patch on the left corner of the top of the bag which is now closest to you. (Figure 1)



Figure 1

Attach the Dricuff, as shown with blue side facing and covering the Velcro patch. Make sure the Dricuff extends beyond the edge of the cover – this will create the watertight seal on the skin. (Figure 2)

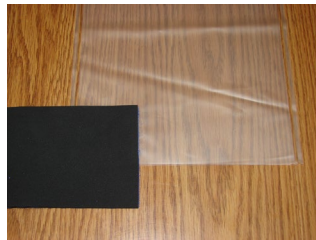


Figure 2

Now put the cover on your arm or leg. (Figure 3) Make a tuck where the Dricuff strap meets the cover, pull the Dricuff strap back onto the cover (Figure 4) and wrap the Dricuff strap around your arm or leg as shown. (Figure 5)



Figure 3



Figure 4

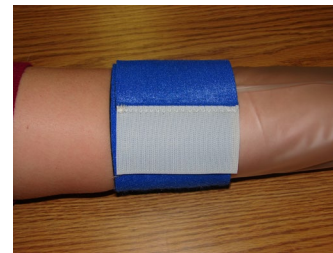


Figure 5

USE ONLY WITH PERMISSION OF YOUR PHYSICIAN

PLASTIC SLIPS EASILY ON WET SURFACES. IT IS RECOMMENDED THAT APPROPRIATE SAFETY STRIPS WHICH MAY DIMINISH THE CHANCES OF SLIPPING BE PLACED ON THE SHOWER FLOOR.